

# (Super Thermagic user manual)

table of Contents

Instruction

Product Icon

System interface icon

Steps

Manipulation

Contraindications

Precautions

Product parameter

Failure and troubleshooting

Instrument maintenance

Packing List

Warranty Card

## 一、 Instruction

Super Thermagic uses high-energy probes to conduct high-energy high-frequency radio waves and HIFU to the depths of the skin. Through three-dimensional heating of deep skin tissues, it can stimulate skin collagen and fibers, reconstruct collagen scaffolds, and make skin tight immediately, continue to stimulate collagen regeneration and maintain long-term lifting effect. It can improve skin sagging, wrinkles, dull complexion and other problems, and keep the face younger.

## 二、 Icon

(设计师插图)


### 三、System interface icon


(设计师插图)

### 四、 Instruction


Skin clean,



Put on the power cord, then press  into control page, Then select the corresponding operation interface according to the operation head you need to use.

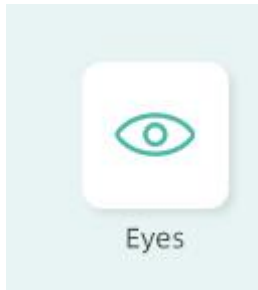

eyes control page,

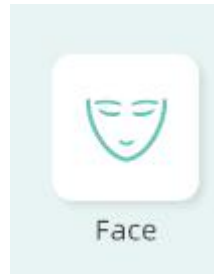

face control page,




anti aging is for HIFU control page.

### Eyes' control page

#### 1、Vibration




Press  vibration , then press



to adjust the vibration intensity.

#### 2、 Energy output time

Press , and then  to adjust the energy output time.


Note: The longer the energy output time, the stronger the energy received by the skin. If the energy is strong, the temperature will be higher. Please adjust according to the temperature that your skin adapts to, otherwise skin damage may occur. It is recommended that 2 seconds is the best for the first operation.

### 3、 Energy intensity output

Press , and then  to adjust the energy intensity output


Note: At the beginning of the operation, it is recommended to start with the lowest temperature, and then adjust it slowly until the most suitable energy intensity for your skin is selected.

### 4、 Tips remaining shots

 shots means the tips remaining shots

### 5、 READY

After setting the vibration,energy output time, and energy output intensity, press

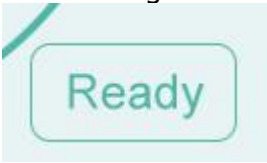, Ready changes color and the instrument enters the ready state, then put the handle on the part to be operated and press the handle switch button to start the operation.

If you need to re-adjust the vibration intensity, energy output, energy intensity, lightly click on ready to release the ready state, you can re-adjust each function.

## Face 操作界面

### 1. Vibration

Press  vibration , then press  to adjust the vibration intensity.

### 2、Energy output time

Press  , and then  to adjust the energy output time.

Note: The longer the energy output time, the stronger the energy received by the skin. If the energy is strong, the temperature will be higher. Please adjust according to the temperature that your skin adapts to, otherwise skin damage may occur. It is recommended that 2 seconds is the best for the first operation.

### 3、Energy intensity output

Press  , and then  to adjust the energy intensity output

Note: At the beginning of the operation, it is recommended to start with the lowest temperature, and then adjust it slowly until the most suitable energy intensity for your skin is selected.


### 4、Tips remaining shots



shots means the tips remaining shots

## 5. READY

After setting the vibration,energy output time, and energy output intensity, press

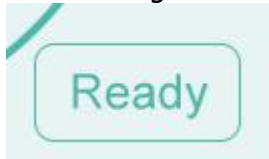



, Ready changes color and the instrument enters the ready state, then put the handle on the part to be operated and press the handle switch button to start the operation.



If you need to re-adjust the vibration intensity, energy output, energy intensity, lightly click on ready to release the ready state, you can re-adjust each function.

## Anti Aging(HIFU) control page


### 1、 ems energy output

Press  slightly, and then press  to adjust the output.

### 2、 HIFU energy output

Press  slightly, and then  to adjust the HIFU output.

### 3. Ready

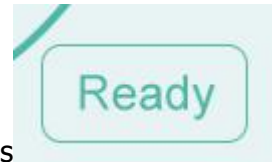

After setting the EMS, HIFU energy intensity output, press ready button turns into dark color, The instrument enters the ready state, then put the handle on the part to be operated, and press the handle switch button to start operation.

If you need to re-adjust the ems energy intensity and hifu energy intensity, tap ready lightly to release the ready state, and then you can re-adjust each function.

#### 4、 Depth

Handle depth adjustment button can adjust the working depth of the



instrument. It is recommended to use 4.5 to operate twice and then 3.0 to operate twice. Only use 3.0 operation on thin skin areas such as forehead and temples.

Note: HIFU operation is not suitable for those who have implanted gold wire within the eye socket, wound site, nose, or face

#### 5.the handle state explanation

When the light above the front of the handle shows blue light, it is in standby state, and red light is in working state.

#### 6. operation product matching

Gel, aloe vera gel, essence, original liquid, thin cream, freeze-dried powder, ampoules, lotion (the operation of the hot Maggie handle cannot be combined with essential oils)

#### 7. instrument operation

Super Thermagic has three operating handles. It is recommended to use the thermagic handle to operate first, and then use the HIFU handle to operate the face.

After the operation, it is recommended to apply a moisturizing mask for 10-15 minutes;

#### 1. Thermagic operation

Neck: From below the collarbone to the top of the skin, it is recommended to avoid the position of the Adam's apple;

Cheeks: Press the "#" shape to operate on the cheeks.

1) Start from the cheek and divide it into three lines from the inside to the outside for fixed-point operation.

Operate from the lower jaw towards the earlobe,

Operate from the corner of the mouth to the direction of the ear,

Operate from the nose to the temple;

Then from the cheek, divide three lines from bottom to top for fixed-point operation.

Operate from the lower jaw to below the cheekbone,

Operate from the middle of the mandible to the direction below the cheekbone,

Operate from the masseter muscle to below the cheekbone.

Finally, from the cheek, it is divided into three lines from the inside to the outside, and the operation is lifted.

Operate from the lower jaw towards the earlobe,

Operate from the corner of the mouth to the direction of the ear,

Operate from the nose to the temple;

Eyes: From the inner corner of the eye socket to the outer corner of the eye socket, use a fixed-point method (it is not recommended to operate the upper eyelid);

Forehead: Operate from the center of the eyebrows to the fixed point next to the hairline, and then from the top of the eyebrows and the tail of the eyebrows to the hairline.

Note:

The operating head should be close to the skin during operation;

The cheekbones, frontal bones, eyes and other areas with less fat have obvious heat sensation. It is recommended to lower the energy.

After each operation, the operating head should be cleaned with warm water or saline.

2. HIFU operation:

Adjust the depth and energy level. At the beginning of the operation, choose 4.5mm depth for operation, and use 3.0 depth for the third and fourth passes;

Test energy intensity: First, pull the right lower jaw to the corner of the mouth at a constant speed. If there is no sensation, you need to increase the mechanical wave/ems energy. It is normal to feel tingling, tingling or soreness during operation. It is necessary to operate it to the nose Lower the energy (adjust the amount of

energy according to personal circumstances), because the position with more bones is more sensitive, and the energy received will be lower (different operating parts, the amount of energy received is also different, you need to adjust the energy according to your own situation size);

Normal skin: The first time you use it, it is recommended that the mechanical wave and ems energy be used at a low level, and the mechanical wave and ems energy should be adjusted to the maximum range that the individual can bear according to the feeling during the operation;

Red bloody skin: For the first use , we suggest only turn on Mechanical wave. After using 3-4 times , you can turn on the RF with the Mechanical wave at the same time, RF energy start from the lowest intensity, and adjust the power of Mechanical wave according to your maximum tolerance .

Sensitive cuticle slants thin skin: For the fist use, Lowest power of Mechanical wave and RF are suggested, After using 2-3 times , Can adjust the energy to your maximum tolerance according to personal skin situation.

6. During the treatment ,it is necessary to apply the medium on the treatment area, the cartridge needs to closed to the skin , with regular speed and lifting method, the medium needs to be supplied when it is absorbed or dried.

7. Usually it takes 30-45 minutes for one operation of the whole face.(The operating time changes according to the size of face. )

#### Operation method

The neck operation: We suggest start from the button line of the neck ,from the clavicle along the neck lifting to jawbone , from the double chin to throat.(Throat area is forbidden ),then from the double chin to lymph gland around, Do remember throat area is forbidden;(Can do operation back and forth 3-4 times, according to personal skin looseness)

The face operation: The intensity start from the lowest, from jawbone to cheekbone, then from mouth to ears, using the lifting manipulation, the last step from the nose side along apple muscle lifting to temple around.(Can do operation back and forth 3-4 times, according to personal skin looseness)

The eye operation: The intensity start from the lowest, from the cheekbone lifting to temple around, then from the after the corner of eye lifting to hairline around, Do remember the upper eyelid and orbital bone are forbidden (Can do operation back and forth 3-4 times, according to personal skin looseness)

The forehead operation: The intensity start from the lowest, from between the eyebrows lifting to hairline around, eyebrows in the middle lifting to temple around, end eyebrows lifting to hairline around, from the button up, from ophryon to temple around, We suggest eyebrows don't do it ;( Can do operation back and forth 3-4 times, according to personal skin looseness)



▪Noted: During the operation, the cartridge cannot stay at one area doing repetitive operation, moving at a constant speed From the very start , we suggest adjusting the intensity from the lowest, then according to the treatment area to adjust the intensity, adjust to the maximum tolerance of the clients, during the operation , you will feel hot, some sensitive area will have the feeling of little sting , after the operation if feel sore and swollen all in normal situation.

▪For normal skin , the first 5 times we suggest three days one time , after the 5 times operation, 5 days one time . For the sensitive skin , the first 5 times , we suggest 5-7 days one time , after the 5 times operation, 10-15 days one time , 8-10 times one treatment. You can design the treatment times according to personal skin looseness.

After the operation may have varying degree's Detoxification metabolism: pimple, acne, lump, flushed, itched, Slight edema, soreness, all in normal situation, usually will recover naturally 1-3 days, in some special case, needs more time for recovery, if that happens , just apply more moisturizing masks,all this cases are normal , tender and temporary feedback reaction after the treatment.

## 八、

People with the following symptoms should be cautious when using this instrument. Please consult a doctor or professional before using it. The details are as follows:

1. If the operating part has a prosthesis, it is recommended to avoid this area when operating the instrument;
2. It is not recommended for women during menstrual period;
3. If there is hyaluronic acid or collagen injection in the operation area in the last six months, please let us know;
4. It is not recommended to operate with a serious heart disease and a heart rhythm regulator;
5. Avoid places such as wounds or purulent acne during operation;
6. It is not recommended to use during skin inflammation or skin allergy;
7. Although it has not been proved that the device is harmful to pregnant women, it is not recommended for pregnant women;
8. The instrument cannot be operated on the eyes, nostrils, ears, mouth, upper eyelids, Adam's apple, etc.;

## 九、

1. Do not wash your face with hot water within three days (you can wash your face with warm or cold water);

2. Do not soak in hot springs or go in or out of hot places within three days;
3. Strengthen the moisturizing and sun protection of the operating parts;
4. It is recommended to apply a moisturizing mask every day within three days of use;
3. Eat less spicy food within three days, smoking, drinking, staying up late is not recommended;

#### 十、 Technical parameters

|                   |                 |            |        |
|-------------------|-----------------|------------|--------|
| Product           | Super thermagic | Model No.  | H-007  |
| Power             | 180W            | Voltage    | 220v   |
| Working voltage   | 36V             | G.W.       |        |
| Package dimension |                 | Frequency: | 1.8~4M |

#### 十一、 Troubleshooting

| Fault  | Troubleshooting   |
|--|---|
| The instrument does not respond after plugging in the power    | <p>First check whether the power strip is powered on, and replace the power strip or socket</p> <p>Check whether the power cord and power connector are loose</p> <p>Check whether the power supply and instrument interface are loose</p>                                |
| Sometimes the instrument cannot be clicked after starting work | <p>Turn off the power and turn it on again. Keep away from the computer and mobile phone during the operation of the instrument. Thermagic is a high-frequency technology product. The magnetic field of the electrical appliance will interfere with the instrument.</p> |

## 十二、 Maintenance

1. The instrument must use a three-pronged plug with a grounding pin, and ensure that the power socket of the instrument is well grounded;
2. Ensure that the power supply is stable and adaptable. If the local power supply voltage is unstable, it is recommended that the user add a regulated power supply with matching power;
3. The instrument should not be placed in a humid place or near a water source, and the instrument should not be directly exposed to sunlight;
4. Keep the instrument away from strong pyrogens or strong radiation sources (disinfection cabinets, induction cookers, microwave ovens, etc.), which will affect the normal use of the instrument;
5. The instrument must use the original qualified accessories provided by the manufacturer;
6. After each operation, the operating handle should be cleaned and disinfected and kept properly;
7. When the instrument is not in use, please turn off the power, unplug the instrument, and place various accessories of the instrument.

To users:

Thank you for your support and trust!

If Super thermagic has any damage caused by the manufacturing process or components within one year of the date of purchase, we promise free warranty (note: man-made damage must be repaired individually)

Warranty

The warranty date within one year starts from the date of purchase;

Please show proof of purchase or purchase record for warranty products;

After the warranty date, repair and replacement parts will be charged at the price of replacement parts;

In one of the following situations, free warranty is not implemented:

The handle and body were dismantled during use, causing the instrument to not be used normally;

Unable to provide the following proof or purchase record.

|            |
|------------|
| Model No.: |
|------------|

|                   |
|-------------------|
| Purchase date:    |
| Client's name     |
| Phone No.:        |
| Fault description |